



Canine Heartworm Antigen Test Kit

Principles

The Anigen Rapid one-step Canine Heartworm Antigen Test Kit is a chromatographic immunoassay for the qualitative detection of Canine *Dirofilaria immitis* antigen in canine serum, plasma, or whole blood.

The Anigen Rapid one-step Canine Heartworm Antigen Test Kit has a letter of "T" and "C" as test line and control line on the surface of the device. Both the test line and the control line in the result window are not visible before applying any samples. The control line is used for procedural control. The control line should always appear if the test procedure is performed properly and the test reagents of the control line are working. A purple color at the test line will be visible in the result window if there are heartworm antigens in the specimen.

The specially selected Canine *Dirofilaria immitis* antibodies are used in the test band as both capture and detector materials. These enable the Anigen Rapid one-step Canine Heartworm Antigen Test Kit to identify Canine *Dirofilaria immitis* antigen in canine serum, plasma, or whole blood with a high degree of accuracy.

Materials provided (10 or 100 Tests/Kit)

- 1) Ten (10) or one hundred (100) Anigen Rapid one-step Canine Heartworm Antigen test devices
- 2) Ten (10) or one hundred (100) disposable droppers for specimen
- 3) Ten (10) or one hundred (100) vials of EDTA
- 4) One (1) Instructions for use

Precautions

- 1) For veterinary diagnostic use only.
- 2) For best results, strict adherence to these instructions is required.
- 3) All specimens should be handled as being potentially infectious.
- 4) Do not open or remove test devices from their individually sealed pouches until immediately before their use.
- 5) Do not use the test device if the pouch is damaged or the seal is broken.
- 6) Do not reuse test device.
- 7) All reagents must be at room temperature (15~30°C) before running the assay.
- 8) Do not use reagents beyond the stated expiration date marked on the label.
- 9) The components in this kit have been quality control tested as standard batch unit. Do not mix components from different lot numbers.

Storage and Stability

The kit can be stored at room temperature or refrigerated (2~30°C). The test kit is stable through the expiration date marked on the package label. **DO NOT FREEZE.** Do not store the test kit in direct sunlight.

Specimen Collection and Preparation


- 1) The test can be performed with serum, plasma, or whole blood (with anticoagulant such as EDTA). When using the included EDTA tubes, you should first collect the blood sample in a syringe and then put into the EDTA tube. Mix this around before using disposable dropper to draw the sample out and drop into the test device.
- 2) If specimens are not immediately tested, they should be refrigerated at 2~8°C. For storage more than 48 hours, freeze the specimen at -10°C or below.

*Note: Blood samples should not be frozen prior to testing.

Procedure of the Test

- 1) Remove the test device from the foil pouch, and place it on a flat and dry surface.
- 2) Draw up the specimen using the disposable dropper.
- 3) Add two (2) drop (approximately 80 µl) of canine serum, plasma or whole blood into the sample hole. **If the migration has not appeared after 1 minute, add one more drop into the sample well.**
- 4) As the test begins to work, you will see a purple color move across the result window in the center of the test device.
- 5) Interpret test results at 5 ~ 10 minutes. Do not decide after 20 minutes.

[Figure for test Procedure]

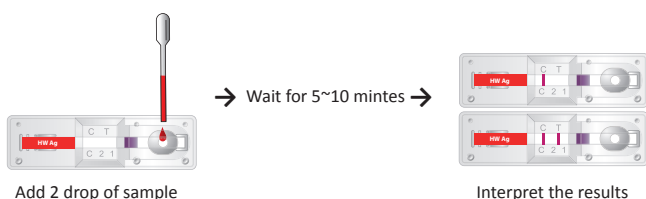


Interpretation of the Test

A colored band "C" will appear in the left section of the result window to show that the test is working properly. This band is the control band. The right section of the result window indicates the test results.


1) Negative result

The presence of only one purple color band within the result window indicates a negative result.

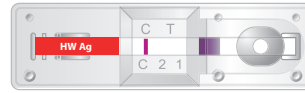


2) Positive result

The presence of two color bands ("T" band and "C" band) within the result window, no matter which band appears first, indicates a positive result.

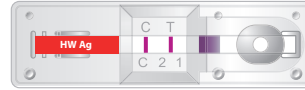

3) Invalid result

If the control purple colored band ("C") is not visible within the result window after performing the test, the result is considered invalid. The directions may not have been followed correctly or the test may have deteriorated. It is recommended that the specimen be re-tested.

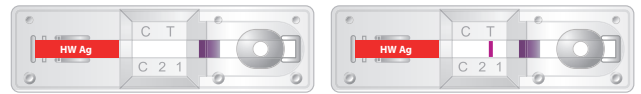

Limitations of the Test

Although the Anigen Rapid one-step Canine Heartworm Antigen Test kit is very accurate in detecting Canine *Dirofilaria immitis* antigen, a low incidence of false results can occur.

Other clinically available tests are required if questionable results are obtained. As with all diagnostic tests, a definitive clinical diagnosis should not be based on the results of a single test, but should only be made by the veterinarian after all clinical and laboratory findings have been evaluated.

In a study conducted in the US, the kit showed a specificity of 100% (54/54) and a sensitivity of 96.4% (53/55). In samples with "2 or more female worms" recovered at necropsy, the sensitivity was 100% (41/41). In a separate multicentric field trial study conducted in the US, the sensitivity (15/15) and specificity (5/5) were found to be 100%.


Bibliography

- 1) Charles H, Courtney and John A. Cornell: Evaluation of heartworm immunodiagnostic test; JAVMA 197(6), 1990, 724~729
- 2) Charles H, Courtney, Qi-Yun Zeng: Comparison of heartworm antigen test kit performance in dogs having low heartworm burdens; Veterinary Parasitology, 96, 2001, 317~322
- 3) William M. O. Boto, Kendal G. Powers and David A. Levy: Antigens of *Dirofilaria immitis* which are immunogenic in the canine host: detection by immune-staining of protein blots with the antibodies of occult dogs; J. Immunol., 133(2), 1984, 975~980
- 4) Matsumura, K., Kazuta, Y., Endo, R., Tanaka, K., Inoue, T., and Matsuda, H: A microassay for *dirofilarial* antibodies by using antigen-coated nitrocellulose paper; Vet. Parasitol., 27, 1988, 337~344
- 5) John P. Thilsted, Jerry Whorton, Clair M, Hibbs, Gregory P, Jillson, Rick Steece and Mary Stromei: Comparison of four serotests for the detection of *Dirofilaria immitis* in dogs; Am. J. Vet. Res., 48(5), 1987, 837~841
- 6) L.C. Wang: Comparison of a whole blood agglutination test and ELISA for the detection of the antigens of *Dirofilaria immitis* in dogs; Ann Trop Med Parasitol, 92(1), 1998, 73~77

Key for symbols on box



In vitro diagnostic

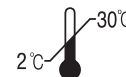

Store between 2~30°C



Read insert for full information

Doc. No.: I1102-04US
Issued date: Feb. 20, 2014



Manufactured by **BioNote, Inc.**
2-9, Seogu-dong, Hwaseong-si, Gyeonggi-do, Korea (445-170)
Tel: +82-31-211-0516 | Fax: +82-31-8003-0618
bionote@bionote.co.kr | <http://www.bionote.co.kr>



Manufactured for
Modern Veterinary Therapeutics, LLC
18001 Old Cutler Road, Suite 633 Miami, Florida 33157 - USA
Tel.: +1 407 852 8039 | Fax.: +1 305 503 8585
info@modernveterinarytherapeutics.com
<http://www.modernveterinarytherapeutics.com>
U.S. Permit No. 544B