## SENSITIVITY AND SPECIFICITY OF CANINE HEARTWORM AG TEST KIT

Cuong Tu Ba, DVM, MS<sup>1</sup>, Catherine Berezaie<sup>2</sup>, Craig Reinemeyer, DVM, PhD<sup>3</sup>,  
Tom Rainey, DVM<sup>4</sup>, Sharon Motta<sup>5</sup>

<sup>1</sup>Modern Veterinary Therapeutics, LLC - Coral Gables, Florida - USA;  
cuong.tuba@modernveterinarytherapeutics.com

<sup>2</sup>Modern Veterinary Therapeutics SAS - Saint-Lunaire - FRANCE

<sup>3</sup>East Tennessee Clinical Research, Inc., Rockwood, Tennessee - USA

<sup>4</sup>Dixie Animal Hospital - Miami, Florida - USA

<sup>5</sup>South Coral Gables Animal Hospital, Coral Gables, Florida - USA

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The **Canine Heartworm Ag Test Kit** is a new *in-vitro* immunochromatographic one step assay for detection of heartworm antigen in canine serum, plasma or whole blood. Four (4) studies were conducted to check the sensitivity and specificity of this new diagnostic test.

In the first study, the **Canine Heartworm Ag Test Kit** showed a sensitivity of 94.4% (68/72) and a specificity of 100% (102/102).

In the second study, the **Canine Heartworm Ag Test Kit** showed a sensitivity of 96.4% (53/55) and a specificity of 100% (54/54).

In the third study, the field robustness of the **Canine Heartworm Ag Test Kit** was evaluated by sending 20 blinded serum samples to three unrelated veterinary sites. The status of each sample was confirmed prior to the shipment to the study sites. Study investigators were requested to perform the test without knowing the status of each sample. Results show a 100% agreement between all study sites. The sensitivity was 100% (15/15) and the specificity was 100% (5/5).

In the last study, the **Canine Heartworm Ag Test Kit** was evaluated for potential cross-reactivity with canine parvovirus, canine distemper virus, canine parainfluenza virus, infectious canine hepatitis virus, porcine parvovirus, *Leptospira icterohaemorrhagiae*, *E. coli* spp., *Salmonella* spp., *Ascaris* homogenates, and *Giardia* homogenates. The results showed that the **Canine Heartworm Ag Test Kit** is highly specific for the heartworm antigen with no cross-reaction with any of the other common canine infectious pathogens.

Data from these studies show that the **Canine Heartworm Ag Test Kit** is highly sensitive and specific for detection of heartworm antigen in serum, plasma or whole blood. Therefore, the **Canine Heartworm Ag Test Kit** is recommended to be adopted in animal clinics for diagnosis of heartworm infection.

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## 2. Introduction

2.1 Introduction – Heartworm (*Dirofilaria immitis*) is a relatively large parasite that, in adulthood, lives in the heart and pulmonary arteries of an infected dog. Dogs acquire this infection when larval heartworms are transferred between infected and healthy dogs via mosquito bites. Geographic areas with high mosquito populations have severe heartworm problems.

2.2 Background - The **Canine Heartworm Ag Test Kit** is an *in-vitro* immunochromatographic one step assay designed for qualitative determination of heartworm antigen in serum, plasma or whole blood.

2.3 Objectives - The primary objective was to determine the sensitivity and specificity of the **Canine Heartworm Ag Test Kit**.

## 3. Study design

3.1 Study 1: One hundred-seventy-four (174) serum samples from clinical cases were collected. Seventy two (72) samples were confirmed positive and hundred-two (102) samples were negative. All samples were then tested by the **Canine Heartworm Ag Test Kit** to evaluate the sensitivity and specificity.


3.2 Study 2: This is a parallel comparative study. One hundred-fifty-five (109) serum samples from clinical cases were collected. Fifty-four (55) samples were positive and fifty-four (54) samples were negative. All samples were then tested by the **Canine Heartworm Ag Test Kit** to evaluate the sensitivity and specificity.

3.3 Study 3: In this study, fifteen (15) positive and five (5) negative canine sera samples were obtained from clinical cases. Samples were divided and shipped to three veterinary sites for blind testing using the **Canine Heartworm Ag Test Kit**.

3.4 Study 4: In this study, the **Canine Heartworm Ag Test Kit** was evaluated for potential cross-reactivity with other canine infectious pathogens: canine parvovirus, canine distemper virus, canine parainfluenza virus, infectious canine hepatitis virus, porcine parvovirus, *Leptospira icterohaemorrhagiae*, *E. coli* spp., *Salmonella* spp., *Ascaris* homogenates, and *Giardia* homogenates.

## 4. Materials & methods:

4.1 The **Canine Heartworm Ag Test Kit** is an *in-vitro* immunochromatographic one step assay designed for qualitative determination of heartworm antigen in serum, plasma or whole blood. The study investigators were instructed to process the samples as indicated in the figure below:

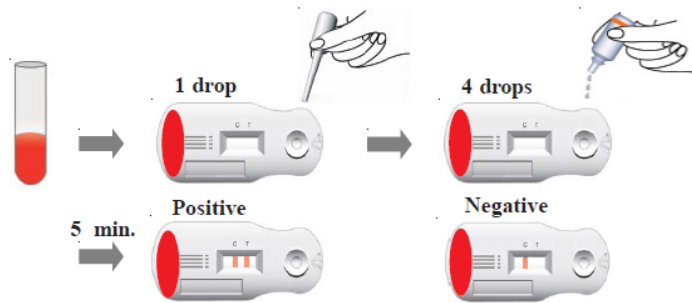4.2 The Interpretation of the test results is conducted as follows:

Negative Result:

Positive Result:

## 5. Analysis

5.1 Sensitivity and specificity will be determined by standard calculation:

$$\text{Sensitivity}(\%) = 100 \times \frac{\text{No. of specimens with positive results by the **Canine Heartworm Ag Test Kit**}}{\text{No. of confirmed positive specimens}}$$

$$\text{Specificity}(\%) = 100 \times \frac{\text{No. of specimens with negative results by the **Canine Heartworm Ag Test Kit**}}{\text{No. of confirmed negative specimens}}$$

## 6. Results:

6.1 Study 1:

Sensitivity and Specificity		Necropsy	
		Positive	Negative
<b>Canine Heartworm Ag Test Kit</b>	Positive	68	0
	Negative	4	102
Sensitivity		94.4% (68/72)	
Specificity		100% (102/102)	

6.2 Study 2:

Sensitivity and Specificity		Necropsy	
		Positive	Negative
<b>Canine Heartworm Ag Test Kit</b>	Positive	53	0
	Negative	2	54
Sensitivity		96.4% (53/55)	
Specificity		100% (54/54)	

6.3 Study 3:

Sensitivity and Specificity			Necropsy		Sensitivity	Specificity
			Positive	Negative		
<b>Canine Heartworm Ag Test Kit</b>	Site A	Positive	5	0	100% (15/15)	100% (5/5)
		Negative	0	15		
	Site B	Positive	5	0	100% (15/15)	100% (5/5)
		Negative	0	15		
	Site C	Positive	5	0	100% (15/15)	100% (5/5)
		Negative	0	15		

6.4 Study 4:

Pathogens	Titer	Results
Canine Parvo virus	10 <sup>4.0</sup> TCID <sub>50</sub> /ml	Negative
Canine Distemper virus	5x10 <sup>3</sup> EID <sub>50</sub> /ml	Negative
Canine Parainfluenza virus	10 <sup>6.0</sup> TCID <sub>50</sub> /ml	Negative
Infectious Canine Hepatitis virus	10 <sup>5.0</sup> TCID <sub>50</sub> /ml	Negative
Porcine Parvovirus	1,024 HAU	Negative
Leptospira icterohaemorrhagiae	OD 1.0	Negative
E. coli spp	10 <sup>8.0</sup> CFU/ml	Negative
Salmonella spp.	10 <sup>8.0</sup> CFU/ml	Negative
Ascaris homogenate	10%	Negative
Giardia homogenate	10%	Negative

7. Discussion and conclusion:

These studies showed that the **Canine Heartworm Ag Test Kit** is highly sensitive and specific for the qualitative detection of heartworm antigen in serum, plasma or whole blood. Using the **Canine Heartworm Ag Test Kit** is very simple, quick and it does not require any special equipments. Therefore, the **Canine Heartworm Ag Test Kit** is recommended to be adopted in animal clinics for diagnosis of heartworm infection.