

# Biogal's ImmunoComb® VacciCheck

## Canine Distemper - Adenovirus Type 2 - Parvovirus Antibody Test Kit, Dot Blot

The Canine VacciCheck is a quick, simple and portable dot blot test kit intended for detection of antibodies to:

Infectious Hepatitis (tested by Adenovirus Type 2 - CAV2)

Parvovirus (CPV)

Distemper (CDV)

### What benefits can the Canine VacciCheck provide?




Canine VacciCheck can:

- Assist in diagnosis of disease status of dogs from one of these diseases.
- Help determine if a dog requires additional vaccination, thus reducing unnecessary vaccination and potential side effects.
- Be useful to determine a dog's unknown vaccination history.
- Determine if puppies have received immunity from initial vaccination program.

### How does the Canine VacciCheck provide such benefits?

The Canine VacciCheck:


- Provides semi quantitative serological results for the three core vaccines.
- Checks these three vaccines in one test in the veterinary clinic or lab.
- Supplies results within 21 minutes.
- Does not require a lab or lab technician to perform the test.

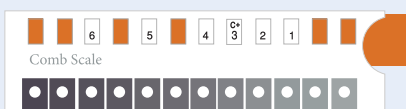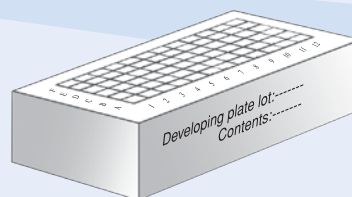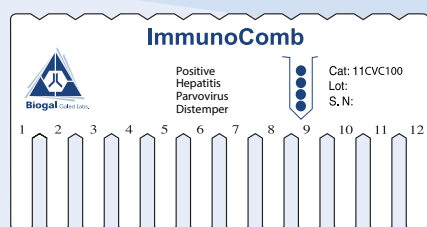


# Canine Distemper - Adenovirus Type 2 - Parvovirus Antibody Test Kit, Dot Blot

## Two Main Components:

The **Developing Plate** has multiple compartments, which contain the necessary reagents for Comb development.

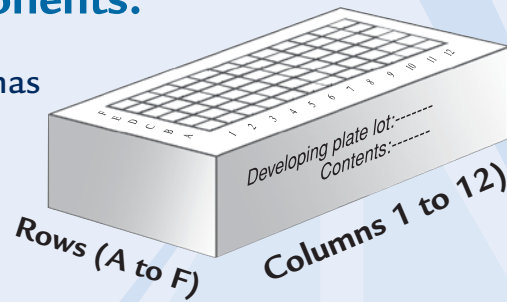

- The **Comb** has 12 teeth.
- One tooth tests one blood/serum sample.
- Each tooth can provide up to three tests: CAV2, CPD and CDV

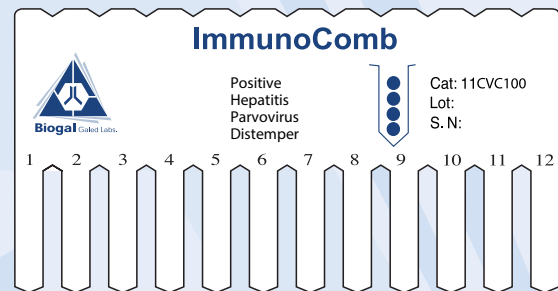

## Kit Development Process:




1 Obtain blood from animal(s).




2 Deposit blood or serum sample(s) into well(s) A in developing plate.



3 Insert Comb into well(s) A. Incubate for a specified time.

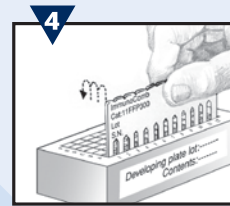

4 Transfer the Comb from well(s) A to well(s) F in a timed sequence. Time intervals vary according to type of Kit.

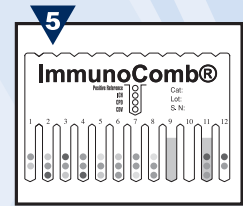

5 Let Comb dry and read results. The color intensity of spots on Comb corresponds to antibody level.

## For Further Information Contact:

### In the USA:

Modern Veterinary Therapeutics, LLC  
 18001 Old Cutler Road, Suite 633 - Miami, Florida 33157 - US  
 Tel: +1 305 234 8228 Fax: +1 305 503 8585  
 e-mail: [info@modernveterinarytherapeutics.com](mailto:info@modernveterinarytherapeutics.com)  
 Website: [www.modernveterinarytherapeutics.com](http://www.modernveterinarytherapeutics.com)